


REF FYL-1S48-100

Rapid Response™ Fentanyl Test Strip

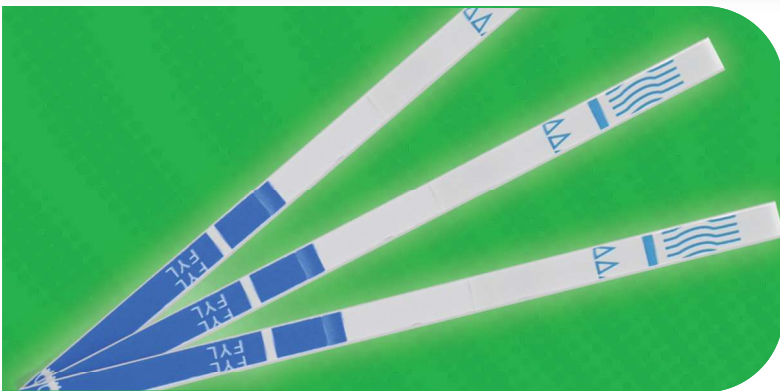

The Rapid Response™ Fentanyl (FYL) Test Strip is an easy-to use, rapid lateral flow assay for the qualitative detection of Fentanyl in urine. It is specific for Fentanyl and does not react to other opiates such as Morphine and Heroin.



Features

- ✓ Individually Packaged Strips
- ✓ Results in 5 minutes
- ✓ Compact, easy to store and transport



<https://bit.ly/3NHpfTH>

John Hopkins Research: Rapid Response™ Fentanyl Test Strips showed the lowest detection limit and highest accuracy among other fentanyl detection methods when tested by John Hopkins Bloomberg School of Public Health.

To learn more about John Hopkins Research, scan this QR code.



<https://bit.ly/3lh6xXa>

VHC: The Vancouver Coastal Health Authority offers Rapid Response™ Fentanyl Strips at their overdose prevention and supervised consumption sites and in their take-home drug testing kits.

To learn more about VHC's Harm Reduction, scan this QR code.


Ordering Information

Product Code	Product Name	Contents
FYL-1S48-100	Rapid Response™ Fentanyl Single Drug Test Strip	100 Tests / Kit



Fentanyl Test Strips

Procedure

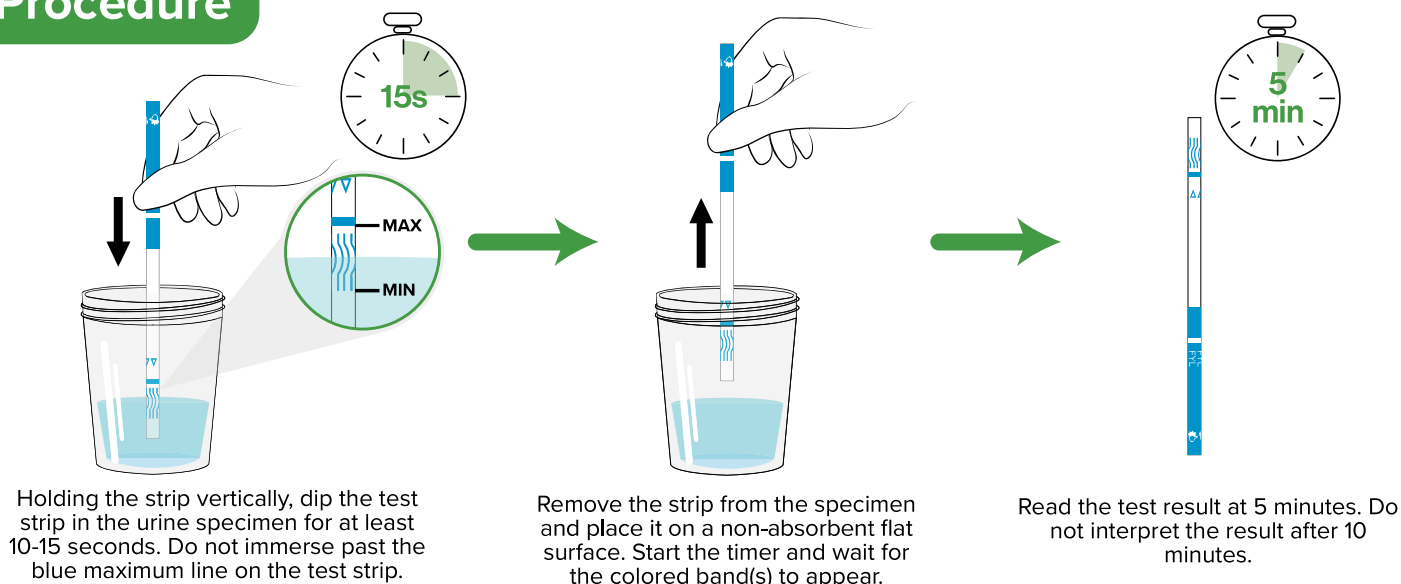


Results


Positive
Contains
Fentanyl



Negative
Fentanyl could
not be detected

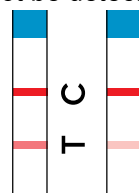

Invalid

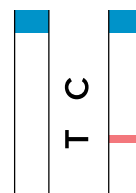

These instructions are for illustrative purposes only. Please read the Instructions for Use supplied with the test before use.

Kit Components

- 100 Individually packaged test strips
- Product Insert

Product Information

- Product Code: FYL-1S48-100
- Sample: Urine
- Format: Strip
- Target: Fentanyl
- Accuracy: > 98%
- Time-to-Result: 5 Minutes
- Storage Condition: 36-86°F/2-30°C
- Test Principle: Lateral Flow Assay

Ordering Information

Product Code	Product Name	Contents
FYL-1S48-100	Rapid Response™ Fentanyl Single Drug Test Strip	100 Tests / Kit

To learn more contact your local **Sales Representative**, call us at **+1 888 339-9964**, or email us at **sales@btnx.com**